Menstrual Cup

MANY THANKS FOR HAVING CHOSEN AMMO

Before using your Ammo cup for the first time, we recommend that you read these instructions to familiarise yourself with it. They are designed to help you from the very beginning.

WHY USE THE MENSTRUAL CUP?

The Ammo menstrual cup is easy to use, safe, hygienic and an excellent alternative to tampons and pads, as it is made of 100% biocompatible medical grade silicone, it does not contain phthalates, and is designed to collect three times the menstrual flow of an ordinary pad or tampon. Depending on your flow, you can enjoy up to 12 hours of comfort without needing to empty it. Our recommendation at the beginning is that you empty it with some frequency until you see what is best suited to your personal case.

Being medical grade silicone ensures complete protection, avoiding allergies and possible adverse reactions to the chemicals and fibres contained in ordinary tampons or pads, as well as toxic shock syndrome. Moreover, the Ammo menstrual cup only collects your menstrual flow, without absorbing the natural fluids from your body that are

essential for your intimate well-being.

As it does not produce waste, the Ammo menstrual cup cares for both you and the environment.

HOW TO CORRECTLY INSERT YOUR AMMO CUP

It is very important that you always thoroughly wash your hands before using it. If you wish, you can also use the Ammo Sanitising Spray. You do not need to sterilise it, but you can do between menstrual cycles or if you have recently had a yeast infection or any other type of vaginal infection.

Follow any of the options we propose below to correctly fold your menstrual cup and ensure that it is completely open inside your body, producing a vacuum against the vaginal walls and thus preventing any leakages.

1- Position yourself comfortably with your legs open. Slightly separate the vaginal lips with one hand whilst slowly inserting the cup with the other, keeping it folded until it slides completely inside the vagina. To ensure a comfortable use, check that the cup is fully open.

2- It is important that the cup makes a vacuum inside your body because if not, you could suffer leakages of menstrual flow, so, once placed, insert one finger and run it along the

edge of the cup to ensure that it has opened correctly, or pull the stem slightly outwards: if it is well placed, it will not move.

USEFUL TRICKS FOR INSERTION

Apply a bit of Ammo moisturiser, or a water-based moisturiser, on to the edges of your cup before inserting it. It will slide in much better and also help it to open correctly.

If you want to be sure that it is still not folded, you can hold the base of the cup and make a half turn.

Do not worry if you do not find it easy to insert. After the first few times, you will slowly find the best position and get the hang of it. It happens to most of us!

To feel more at ease regarding possible leakages, you can use a panty liner the first few times. You will soon discover that you don't need it.

HOW TO REMOVE YOUR AMMO MENSTRUAL CUP

Before removing it, it is very important that you thoroughly wash your hands. This is the most important step to avoid any possible infections.

1- Separate your legs so that you can comfortably insert your fingers into your vagina.

2- Take hold of the body of the cup, press with one finger to release the vacuum, and slowly slide it down in a zig-zag and out of your vagina. This step is essential, because if you pull it without having broken the vacuum, the cup will not remove easily.

3- Empty the contents of the cup. If you can, wash it with soap and water or with the Ammo Sanitising Spray. If not, clean it with a bit of paper. As long as your hands are clean, there will be no problems.

4- Reinsert your cup, following the steps we have already explained above.

At the end of your period, thoroughly clean your Ammo cup before storing it. Never share your menstrual cup.

USEFUL TRICKS FOR REMOVAL

Don't become overwhelmed if it takes a while to remove it the first time. Relax, move your vaginal muscles a little and take hold of the cup with your fingers to remove the vacuum seal.

If you cannot remove it after several attempts, perform a squat. This position presses the vagina, which helps to push the cup outwards. From there, repeat the steps we have already indicated.

As it is made of flexible medical silicone, the Ammo cup will not cause any internal damage.

GENERAL INFORMATION

The menstrual cup is not a contraceptive and should not be used during intercourse.

Do not use it with Vaseline, oils or silicone lubricants as these may damage it.

Always use water-based moisturisers or lubricants.

If you notice any adverse reaction, irritation or pain whilst using your menstrual cup, remove it and consult your doctor.

Made of 100% biocompatible medical silicone. Does not contain phthalates.

Capacities: S size: 20ml / M size: 25ml / L size: 40 ml. Includes a bag for storage.

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