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AMMO EXTENDS

THANK YOU FOR CHOSING AMMO

Before you start using your Ammo Fit Extends for the first time, we recommend that you read these usage recommendations so that you can become familiar with them. They are designed to help you from the beginning.

WHAT ARE AMMO EXTENDS?

Extends are silicone vaginal dilators with progressive therapy vibration-transmitting capacity that complement the treatment prescribed by a health professional, and can

help prevent and achieve an improvement in the symptoms of a series of dysfunctions related to the pelvic floor as they can be:

- Pain during sexual intercourse or deep dyspareunia, vaginitis (inflammation of the vaginal mucosa), endometriosis.
- Treatment after gynecological surgery
- Loss of elasticity in the tissues of the vagina after radiotherapy treatment.
- After childbirth. To work on sensitivity alterations, awareness, scar work (episiotomy or tear) and training.
- Muscle spasm in the pelvic floor due to excessive tension or due to a hyperactive pelvic floor. Vaginismus.
- Pelvic floor pathologies such as: Chronic pelvic pain, pelvic congestion syndrome.
- Vaginal atrophy and/or other menopausal symptoms,
- increased blood flow, improvement of the vaginal mucosa.
- Vaginal narrowing

MEASURING RULER

Apart from progressing the treatment in terms of improving the diameter of the vaginal canal, it is also important to improve its extension, especially in cases of pelvic surgery. In these situations, normally, the pelvic floor becomes more rigid, less flexible and therefore, sexual relations or other types of explorations are more difficult. With the EXTENDS measuring ruler, progress in treatment can be measured in a real way.

HOW TO USE THE EXTENDS WITHOUT VIBRATION

The use of Ammo Extends must always be prescribed by a healthcare professional after carrying out a personalized diagnosis and it will be this professional who will record the evolution and adapt the guidelines for time of use, frequency, etc.

The indications that you will find here are for standard use, consult your doctor if you have any questions:

Select the one with the smallest size and thickness. The use of EXTENDS must be progressive and appropriate to the objective of each user. Dilatation should begin from the smallest dilator, increasing the size gradually, respecting its limits and in a comfortable way.

Wash the Ammo Extend you are going to use with warm water and soap and dry it completely.

Start the exercise:

- 1- Get into a position that is most comfortable, for example, the recommended position is lying on your back with your knees bent and apart. You can also do it standing with one foot elevated with a chair or stool; or squatting. The important thing is to be in a comfortable position and as relaxed as possible.
- 2- To insert the dilator, it is recommended to use a water-based intimate moisturizer to facilitate its insertion. Breathe slowly. Start by gently inserting the tip of the dilator, going as deep as is comfortable, without feeling pain, otherwise you are applying too much pressure. It is normal to feel a stretching sensation.
- 3- It is not advisable to force the introduction of the dilator. It is important for success with EXTENDS to ensure that its use is comfortable and does not cause pain.
- 4- Keep the dilator in the vagina or at its entrance for 2 minutes.
- 5- Slide the EXTEND in and out gently while pressing the vaginal walls, without removing it completely from the vagina.
- 6- Massage the area in a circular motion, moving back and forth, as well as to the sides, looking for points of greatest tension.
- 7- Maintain gentle pressure on the points of greatest tension for a maximum of one minute.
- 8- At the end of the dilation exercise, remove the EXTEND completely and remain relaxed for a few minutes.

It will be time to progress to the next dilator size when you don't feel any discomfort using your current one. This is when your vagina has dilated enough to be able to use the next size.

HOW TO USE EXTENDS WITH VIBRATION

EXTENDS can be used as an extension of the FIT AMMO to take advantage of the benefits of vibration.

Consult your doctor whether to use them with or without vibration. In case of vibration use, insert the Fit into the back of each Extend (as appropriate to the part of the program you are in) and activate the Fit on its button on the back (you must follow the Fit's instructions to its operation).

PRECAUTIONS AND CONTRAINDICATIONS

These are the precautions you should take when using the Ammo Fit Extends:

- Avoid the use of vaginal dilators in case of vaginal or pelvic infection.
- Never force your body until you feel pain. You must notice pressure but not pain.
- Do not continue with the use of vaginal dilators and consult your specialist if blood, pain or a sensation appears after use, excessive vaginal discharge.
- After radiation treatment or obstetric surgery it is strictly necessary to follow medical guidelines in As for when to start this therapy complementary, its frequency and duration.
- Carefully clean and dry your vaginal dilators before and after each use.

To conclude, insist on the need to go to a medical, professional, gynecologist, midwife, physiotherapist or sexologist specializing in pelvic floor to get the right particular diagnosis and personalized treatment, Well, the results can be very positive.

CONTRAINDICATIONS

If you notice any adverse reaction, irritation, pain or discomfort during the use of the Extends, interrupt its use and consultation with your doctor. If you are pregnant, consult your doctor before using it. Keep out of the reach of children and pets.

CLEANING AND MAINTENANCE

Being made of medical grade silicone, Extends are completely safe for your body.

They are intended for use by a single patient.

We recommend that before and after each use, you clean them with warm water and neutral soap. Before storing the dilators in the bag, check that they are dry.

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